

# Super-Snap SuperBuff

CLICK / SCAN



FOR MORE INFO

## IMPREGNATED FELT DISKS FOR SUPER POLISHING DIRECT RESTORATIONS

Designed for clinicians who prefer the convenience of achieving a high gloss super polish without the mess of polishing pastes

- o Single use, snap-on, aluminum oxide impregnated felt disks
- o Fast, convenient and comfortable for the patient
- o Ideal to polish natural teeth after scaling

Super-Snap SuperBuff felt disks are designed for single use and can be used with the same CA mandrel as the Super-Snap disk system. Proven to provide a high lustre super polish similar to natural enamel for enhanced



### Available in:


- **SuperBuff Set (PN0535)**  
*25x unmounted polishers and 2 mandrels (CA shank)*
- **Refills**  
*50x unmounted polishers [PN0536]*  
*6x CA Mandrels [PN0439]*

Super-Snap  
SuperBuff



PN0536

Super-Snap  
Mandrel CA

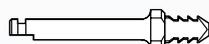

PN0439


## CLINICAL APPLICATIONS

- High gloss polishing of all tooth coloured direct restorative materials
- Polishing after routine scaling

## FEATURES

- ✓ **Mess -free and convenient to use**
  - Snap-on impregnated felt disk
  - Soak in water for 30sec before use
  - Same Mandrel as Super-Snap system
- ✓ **Fast, safe and predictable super polishing**
  - Thick disk design enables super polishing of occlusal surfaces efficiently
  - Soft, safe, elastic silicone snap-on for easy placement on mandrels
  - No metal centre to completely eliminate the risk of gouging restoration
- ✓ **Dry polishing process**
  - Low speed (max 15.000 rpm) with uniform and feather light pressure
- ✓ **Disposable disks** for enhanced infection control

## INSTRUCTION FOR USE

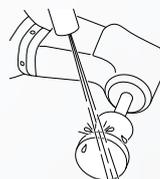

Wet the surface to be polished and moisten the SuperBuff disk with water

## RELATED PRODUCTS

- OneGloss 2-in-1 Smart Polisher
- Super-Snap X-Treme Disk System
- Beautifil II / Beautifil II LS/ Beautifil Injectable X



Official Partner

